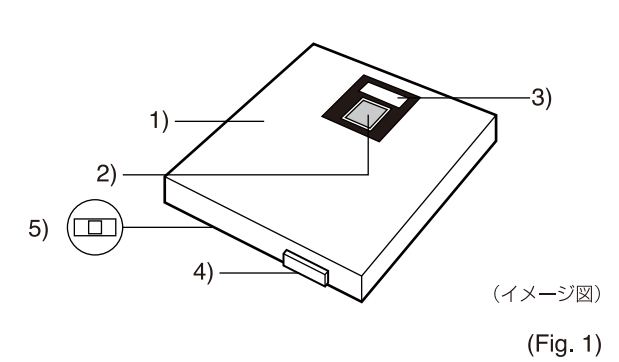
Solar Scale

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- Body
- Solar Cell
- Display
- Reset Switch
- Mode Switch

2. Before use

Place your solar scale onto a flat, hard surface, while being exposed directly to room or natural light. This scale needs a certain brightness of light in order to operate. If the light available is dimmer than required, the display may show “LO”. In this case, move the scale to a brighter place. If the room is not bright enough, then there is a possibility that the scale may not even show “0.0”, or “LO”.

3. Instructions for setting up the scale

As the power is always on simply step onto the scale to weigh. If the scale is used for the first time or if the scale has just been moved to a new location, tap the reset switch and make sure the display shows “0.0” before stepping on.

- Check that the display shows “0.0”. If the display shows anything other than “0.0”, tap the reset switch.
- Step on the scale. The display shows body weight.
- Step off the scale, the display returns to “0.0”. If the display returns to anything other than “0.0”, the previous reading may be wrong. Tap the reset switch, and step on again.

4. Caution

- When using the reset switch, only gently tap the switch as illustrated (Fig. 2). If your hands come into contact with the scale whilst it is resetting then this may display “-0.2” or “-0.4” (Fig. 3), if this happens please tap the reset switch again and wait for “0.0” displayed.
- Do not prop the scale up against a wall with the reset switch facing down (Fig. 4).
- Do not place the scale with the reset switch in contact with a wall or other object (Fig. 5).

5. Features

This bathroom scale is solar-powered, and should have ample power as long as it is exposed to an adequate level of light (Fig. 6). When the display shows “0.0”, it is ready to weigh. Simply step on the scale to weigh.

- Switching the Mode Switch
Select your preferred weight mode using the weight mode switch on the bottom of the scale.

6. Care and Instruction

If the display shows anything other than “0.0”, tap the reset switch. Take care not to spill water or chemical liquids on the scale. Wipe the scale with a water dampened cloth periodically. “OL” will appear when weight surpasses the scale’s capacity.

7. Specification

Version	kg	lb	st-lb
Capacity	150 kg	330 lb	23 st 8 lb
Min. Grad.	3-100 kg/100 g	6.6-200 lb/0.2 lb	6.6-10 st/0.2 lb
	100-150 kg/200 g	200-330 lb/0.5 lb	10 st-23 st 8 lb/0.5 lb
Power	Solar cell	Solar cell	Solar cell

Federal Communications Commission and Canadian ICES Notice

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules and Canadian ICES-003.

These limits are designed to provide reasonable protections against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio or television technician for help.

Modifications

The FCC requires the user to be notified that any change or modifications made to this device that are not expressly by Tanita Corporation may void the user’s authority to operate the equipment.

CE	This device meets the EMC Directive 2004/108/EC and the RoHS Directive 2011/65/EU.
-----------	--

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© 2009-2018 TANITA Corporation HS3027631(1)-1801FA